

NITRIC OXIDE REGULATOR

FLOTEC INC

Web Site: <http://www.floteco2.com>

7625 West New York Street 317-273-6960 Fax 317-273-6979
Indianapolis, Indiana 46214-4911 e-mail: flotec@ix.netcom.com

Order desk: 800-401-1723
Order desk Fax 800-515-9254

The FLOTEC nitric oxide regulator is designed to provide specified flows at approximately 50 PSI. The FLOTEC regulator is preset to deliver the specified flows using pure nitrogen. Every FLOTEC regulator design has an internal safety relief valve to prevent overpressurization of the regulator. FLOTEC products contain no latex or latex byproducts.

SPECIFICATIONS:

Operating Pressure: 250-3000 PSI
Outlet Pressure: Nominal 50 PSI
Inlet Configuration: CGA 660 Nut & Nipple
Outlet Configuration: Hose Barb

REPLACEMENT PARTS:

CGA 660 Nut & Nipple
CGA 660 Teflon Washer
Stainless Steel Hose Barb
Vent Port Hose Barb (s)
Stainless Steel Gauge

INSTALLATION: CGA 660 Connector:

When connecting the CGA 660 connection use the appropriate wrench to tighten the nut. Tighten further if leakage occurs at the CGA 660 connection. If the CGA 660 connection continues to leak, examine and replace the CGA 660 teflon washer as needed.

INSTALLATION REGULATOR:

The Flotec regulator has an internal safety valve to prevent overpressurization of the regulator. Point of installation, local and state regulations may require venting of the relief valve to outside of the use/storage area.

WARNING: If the internal safety relief valve should open because of a malfunction, large quantities of gas can be released. Nitric oxide can be dangerous. Large quantities of nitric oxide can push the oxygen out of a room. A room filled with nitric oxide can cause a person to faint and/or asphyxiate. Consult CGA standards for other cautions.

PREVENTIVE MAINTENANCE: All nitric oxide regulators should be tested and cleaned periodically to insure proper performance. The frequency of testing should be established according to usage, but it should be performed at least once per year to evaluate for damage, contamination, wear and performance.

LEAK TEST: Attach the regulator to an appropriate cylinder, plug all outlets, turn the flow selector to "OFF" and slowly open the cylinder valve. Apply a compatible leak test solution to all outlets, fittings, and the junction of the gauge and regulator body. Tighten fittings as required to eliminate all external leaks. DO NOT overtighten threaded connections. Replace CGA 660 washer if required. Close cylinder valve.

WARNING:

Nitric oxide at room temperature is a colorless, nonflammable, toxic gas. Nitric oxide in the presence of air forms brown fumes of nitrogen dioxide, which is extremely reactive and a strong oxidizing agent.

Nitric oxide, with the attendant formation of nitrogen dioxide, results in a strong respiratory irritant which may be fatal.

When Nitric oxide is in the presence of moisture and oxygen, corrosive conditions will develop as a result of the formation of nitric and nitrous acids, therefore aluminum or stainless steel fittings and components should be used, do not use brass fittings or components.

Disassembly, assembly, and testing of regulators should be performed only by trained personnel. The work area must be free of hydrocarbon contaminants and residues because of the danger of spontaneous combustion when residues are exposed to gaseous nitric oxide/nitrogen mix.

SAFETY WARNINGS:

1. The use of FLOTEC nitric oxide regulators for gases other than compressed nitric oxide is expressly prohibited by FLOTEC and the user must assume all liabilities.
2. Storage areas should be dry, cool and well ventilated. Where practical, storage areas should be fireproof. Storage in sub-surface locations should be avoided. Storage conditions should comply with local and state regulations.
3. Never administer oxidizing gases when smoking or when near an open flame.
4. Never use medical gases from a cylinder without reducing the pressure through a suitable regulator intended for that gas.
5. Make sure that the threaded fittings on regulators are properly mated for the gas intended. Never attempt to force an incompatible connection.
6. Never permit compressed medical gases to enter a regulator suddenly. Always open the cylinder valve slowly.
7. Fully open the cylinder valve when a regulator is attached and in use.
8. Never leave a cylinder valve open with regulator attached when regulator is not in use.
9. Before a regulator is removed from a cylinder, fully close the cylinder valve and release all gas from the regulator.
10. Never interchange regulators, hoses, or other equipment with similar equipment intended for use with other gases. Pressure regulators and related fittings should never be handled with oily or greasy hands or gloves. Never hold hand over the outlet(s) to test for the presence of pressure.
11. Never use nitric oxide as a pressure medium to purge obstructed pipelines or equipment, to operate pneumatic tools, or to build up pressure in any tanks.
12. Do not stand in front of a regulator outlet when opening the cylinder valve in case foreign particles are present which could cause a hazardous malfunction of the regulator.
13. Nitric oxide therapy may be critical treatment. The application of the regulator should be made in strict accordance with the prescription and instructions of a physician.
14. Secure cylinders to wall, stand, or cart in accordance with local fire codes.
15. The regulator is equipped with an internal safety, which is designed to protect the regulator. Downstream equipment used in conjunction with the regulator should be equipped with suitable safety valves to prevent overpressurization and damage.
16. Nitric oxide must never be allowed to contact oil, grease or other petroleum-based substances. Therefore, do not use oil or grease on nitric oxide regulators, cylinders, valves or other related equipment. Do not use or store nitric oxide equipment near excessive heat (>125 F or 51.5 C) or open flame.
17. CAUTION: Do not use organic-based threaded sealants on any portion of the regulator. Use only Teflon threaded-sealing tape or nitric oxide service thread compound.
18. Never leave pressurized nitric oxide within a regulator. Always purge residual gases when not in use.
19. The regulator must be tight to prevent leaking around the CGA 660 Nipple and those using a tip seal or face washer.
20. The regulator must NOT be twisted or turned when it is tight on the cylinder valve and/or under pressure to prevent tip seal, or face washer abrasion.

Flotec Patent #'s: 4909476, 5785050, 6026854, 6082396, 6328280, 7089956, 4982760, 6148841, 7255127

FLOTEC INC

7625 West New York Street
Indianapolis, Indiana 46214-4911

317-273-6960
Fax 317-273-6979

Web Site: <http://www.floteco2.com>
E-mail: flotec@floteco2.com

Order desk: 800-401-1723
Order desk Fax 800-515-9254

Declaration of Conformity

Product identification

Product name : Regulators & Flow meters, Regulators with Flow meter.
Brand : Flotec
Cat. Number : RXXXX-XXXXX, FXXX-XXXXX, MXXXX-XXXXX, DXXXX-XXXXX
Batch/Serial Nr. : Numbered Sequentially

Manufacturer

Name : Flotec
Address : 7625 West New York Street
Indianapolis, IN 46214-4911
Country : USA
Representative : Brian Davidson
Tel : +1 317-273-6960
Fax : +1 317-273-6979

Authorized Representative in Europe

Name : Emergo Europe
Address : Molenstraat 15
2513 BH
The Hague
Country : The Netherlands
Tel : (31) (0) 70 345-8570
Fax : (31) (0) 70 346-7299

Notified Body

Name : UL International (UK) Ltd
Address : Womersley House, The Guildway
Old Portsmouth Road
Country : UK-Guildford, Surrey, GU3 1LR
I.D. Number : CE 0843

Means of conformity

Flotec Inc. declares that the product listed has been classified as Class IIb - Annex IX, Rule 11 and is in conformity with the essential requirements and provisions of Council Directive: 93/42/EEC

Signature

Place and date : Flotec Inc 9-22-03

Signature : 

Name : Brian Davidson
[Name and Signature of authorized person within organization]

FLOTEC INC

7625 West New York Street
Indianapolis, Indiana 46214-4911

317-273-6960
Fax 317-273-6979

Web Site: <http://www.floteco2.com>
E-mail: flotec@floteco2.com

Order desk: 800-401-1723
Order desk Fax 800-515-9254




Authorized Representative

Name : Emergo Europe
Address : Molenstraat 15
2513 BH The Hague
Country : The Netherlands
Tel : (31) (0) 70 345-8570

Symbols

Symbols

PSI
KG/CM²
BAR
MPA
HP
LPM
SN

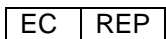

Definition

POUNDS PER SQUARE INCH
KILOGRAMS PER SQUARE CENTIMETER
KILOGRAMS PER SQUARE CENTIMETER
MEGA PASCAL
HIGH PRESSURE PORT
LITERS PER MINUTE
SERIAL NUMBER

USE NO OIL

CONSULT INSTRUCTIONS FOR USE

AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY